3SHAPE

EXOCAD

DENTAL WINGS

DESS LAB LIBRARY INSTALLATION

FOR CUSTOM ABUTMENTS DIRECT TO IMPLANT

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INS-13-MUL-01

08-09-2016

3Shape DESS LAB ONE PIECE CUSTOM LIBRARY_[Version]

ENGLISH

- 1 • Unzip the folder "3Shape DESS LAB ONE PIECE CUSTOM LIBRARY_[Version].zip". This folder contains the file "3Shape DESS LAB ONE PIECE CUSTOM LIBRARY_[Version].dme", the components of library DESS LAB for 3Shape.
- 2 • Open the Dental System Control Panel and go to "Tools" → "Import/Export".
- 3 • Choose "Import materials". Windows Explorer opens and you need to choose the unzipped .dme file.
- 4 • The DESS LAB Library is ready to be used.

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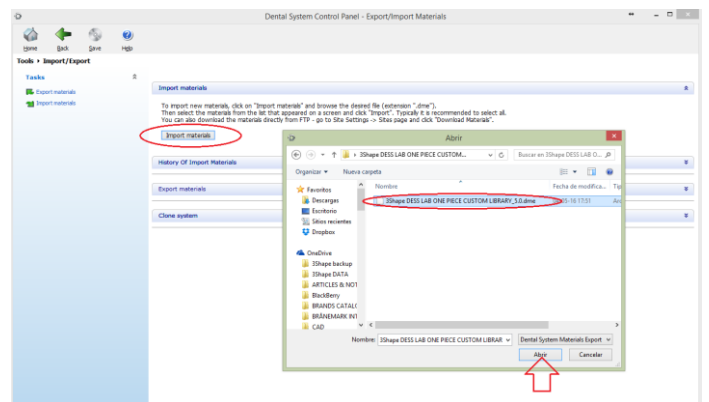
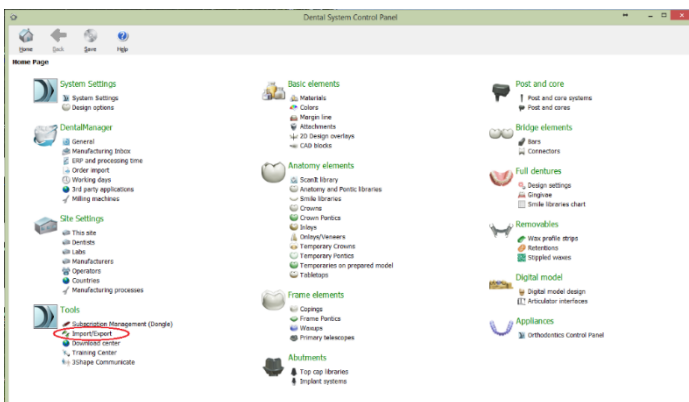
Exocad DESS LAB ONE PIECE CUSTOM LIBRARY_(Version)

ENGLISH

- 1 • Copy the file "Exocad DESS LAB ONE PIECE CUSTOM LIBRARY_[Version].7z" after the download and paste it in the index for the Exocad libraries: DentalCADApp\library\implant
- 2 • Unzip the file in the same index: DentalCADApp\library\implant
- 3 • Delete, if desired, the file "Exocad DESS LAB ONE PIECE CUSTOM LIBRARY_[Version].7z" from your hard drive.
- 4 • The DESS LAB Library is ready to be used.

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DWOS DESS LAB ONE PIECE CUSTOM LIBRARY_(Version)

ENGLISH

- 1 • Start the program Dental Wings and choose the module "Implant Kits".
- 2 • On the chosen module "Implant Kit Editor" do a right mouse click. Choose with a left mouse click the menu item "Import Implant Manufacturer Kits".
- 3 • Choose the file "DWOS DESS LAB ONE PIECE CUSTOM LIBRARY_(Version).zip". The loading of the folder can take a few minutes.
- 4 • The DESS LAB Library is ready to be used.

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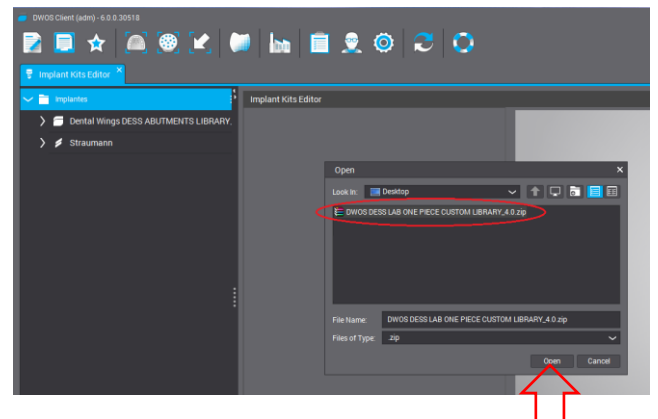
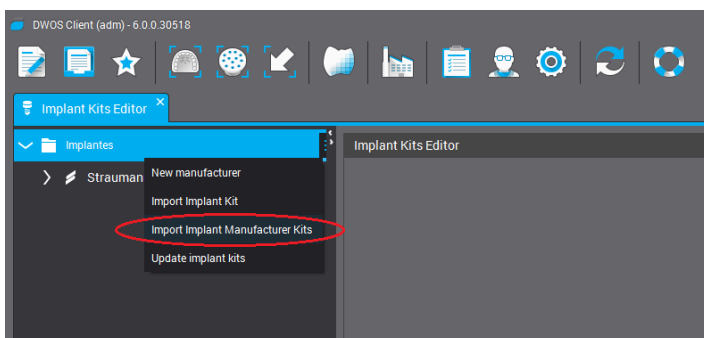
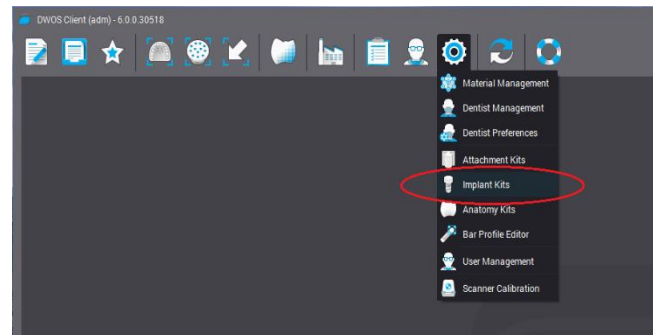
These instructions replace all previous editions. For any detailed information about DESS LAB® products please contact your local distributor. The photographs of the products are for information purposes only. The type, reference and connection of each element should be checked. All DESS LAB® products are manufactured according to ISO 9001:2008 and ISO 13485:2012. | 3i® and 3i® Certain®, Miniplant®, MicroMiniplant™ and Osseotite® are registered trademarks of 3i Implant Innovations, USA | ASTRA TECH™, OsseoSpeed™, ANKYLOS® C/X, XIVE® and FRIADENT™ and UniAbutment™ are registered trademarks of DENTSPLY Implants | BioHorizons® registered trademark of BioHorizons, INC. | Brånemark System®, NobelReplace® and Replace Select™ and NobelActive™ and NobelReplace® CC and NobelSpeedy™ and Multi-unit® registered trademarks of Nobel Biocare AB, Sweden | ITI®, Straumann® and synOcta® registered trademarks of Straumann Holding AG, Switzerland | MIS® registered trademark of MIS Implant Technologies Ltd | Zimmer Screw-Vent® and SwissPlus® registered trademarks of Zimmer Dental INC, USA | Dyna® registered trademark of Dyna Dental Engineering BV | Locator® registered trademark of Zest Anchors INC. | 3Shape® is a registered trademark of 3Shape A/S | EXOCAD® is a registered trademark of Exocad GmbH, Germany | Dental Wings® and DWOS® are registered trademarks of Dental Wings, Inc.